

# ASSISTED COLLECTION KIT

Your assisted collection kit consists of **2 separate devices which**, in conjunction will enable you to collect samples from persons who would otherwise not be able to produce the required amount of saliva.

Follow the instruction below if the sample donor is **not able to spit**.

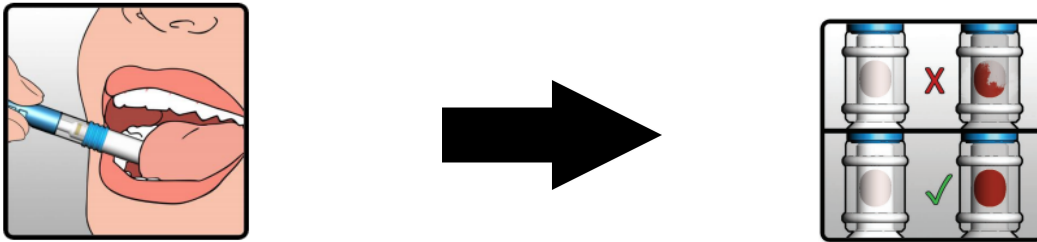
**Sample donors should not eat, drink smoke or use oral hygiene products for at least 30 minutes before you start the collection process.**

**For difficult sample donors, it is advised to collect samples while they are asleep.**

## STEP 1

**Use the Super•SAL™ device to collect your saliva sample:**


Pull the white absorbent pad out of the syringe cylinder and place the white absorbent collection pad end of the Super•SAL™ device in the mouth, along the side of the tongue and collect until the appearance of the Sample Volume Adequacy Indicator [SVAI] changes from **white to fully red**.



It is appropriate to remove the device from the mouth periodically in order to read the SVAI, but immediately return the collection device to the mouth and resume collection. The collection procedure should last approximately 1-3 minutes. Ensure that sample donor does not chew or suck on the absorbent collection pad.

## STEP 2

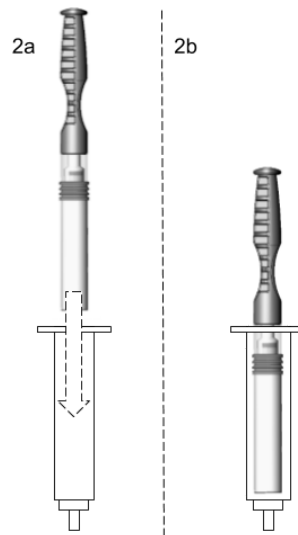**Place the white absorbent pad end into the Plastic Compression Tube holding the Super•SAL™ device in an upright and vertical position.**




To view PDF version of this instruction sheet, go to [www.mthfr-genetics.co.uk/instructions.html](http://www.mthfr-genetics.co.uk/instructions.html)

# ASSISTED COLLECTION KIT
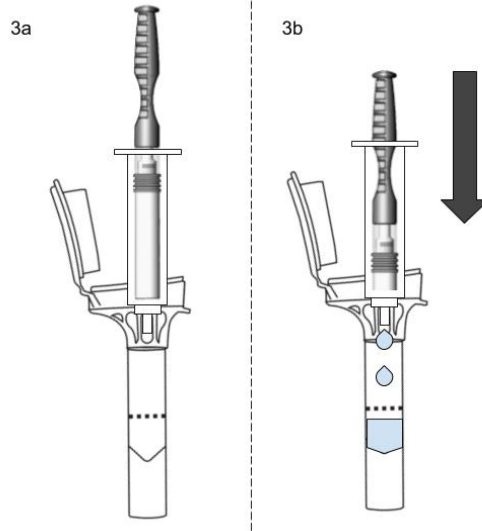
## STEP 3

Insert the tip of the Super Sal syringe inside the funnel on top of the Oragene collection tube. Firmly push the plunger downwards to transfer saliva from the absorbent pad into the Oragene Saliva Collection Tube. Push and hold for 30 seconds.







FILL THE ORAGENE COLLECTION TUBE AS CLOSE AS POSSIBLE TO THE "FILL TO" LINE.

REPEAT STEPS 1 THROUGH 3 IF NEEDED

CAUTION: YOU MAY HAVE TO USE CONSIDERABLE FORCE TO PULL PLUNGER WITH THE ABSORBENT PAD BACK OUT OF THE SYRINGE FOR REUSE. THE RED INDICATOR WILL NOT RESET.

## STEP 4

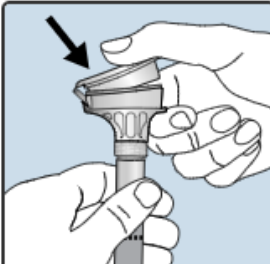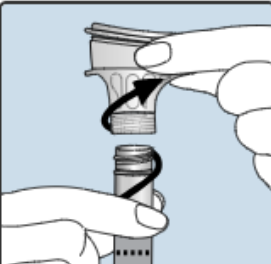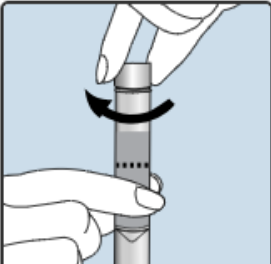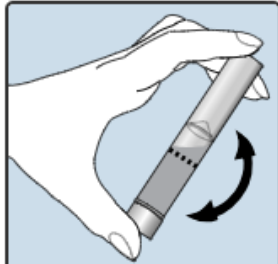Finish collection by following instructions for the Oragene collection tube, below:

			
<p>1. Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.</p>	<p>2. Hold the tube upright. Unscrew the funnel from the tube.</p>	<p>3. Use the small cap to close the tube tightly.</p>	<p>4. Shake the capped tube for 5 seconds. Discard or recycle the funnel.</p>

To view PDF version of this instruction sheet, go to [www.mthfr-genetics.co.uk/instructions.html](http://www.mthfr-genetics.co.uk/instructions.html)